

Surecheck[®] Streamer



Pregnancy test with over 99% accuracy.
Results as early as day one of missed period!

User information leaflet
Please read the instructions carefully before use!

The prospect of being pregnant can bring joy or anxiety. In either case, the best thing to do if in doubt is to find out for sure. Surecheck[®] is developed with the special Colloidal Gold technology to ensure test result accuracy above 99% from day one of the missed period – at any time of the day. The specially engineered sleek design makes Surecheck[®] more hygienic and easy to use.

Accurate result each time, anytime!


ACCURATE AT ALL TIMES

Store at a temperature between 4°C and 30°C.

Simple to use, no special or previous training required for taking the test.

Before you begin **1.** Read the instructions carefully. **2.** Have a watch, clock or timer handy. **3.** Check the expiration date on the box. Do not use if expired. **4.** Do not use the test in case of foil pouch damage.

HOW TO USE

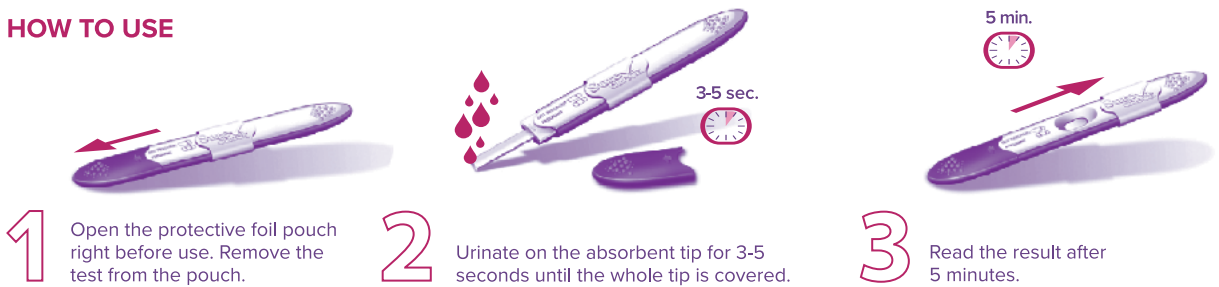


1 Open the protective foil pouch right before use. Remove the test from the pouch.

2 Urinate on the absorbent tip for 3-5 seconds until the whole tip is covered.


3 Read the result after 5 minutes.

INTERPRETATION OF RESULTS

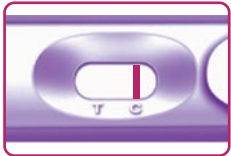

NOT PREGNANT

A single coloured line appears in the control zone (C).

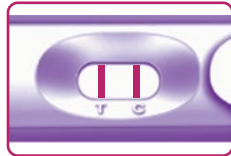

PREGNANT

A single coloured line appears in the control zone (C). Another single colored line appears in the test zone (T).

Invalid result

- If no line appears in the control zone (C) after 5 minutes, the result is invalid. Repeat the test using a new Surecheck[®] Streamer test kit.
- The test is rendered invalid if it is removed from the foil pouch 15 minutes or more before use.



- Disregard any changes in the test result after 30 minutes.
- Surecheck[®] Streamer should not be reused!

Consult a doctor in case of positive result!

Consult a doctor before any further actions!

FAQ

1. How does Surecheck[®] work?

Surecheck[®] registers the presence of hCG (human Chorionic Gonadotropin), a hormone, in the urine. hCG appears in the urine soon after conception. When urine containing the hCG hormone interacts with the pregnancy test, two lines appear in the result window.

2. When can I use the test?

Surecheck[®] can be used as early as the first day of a missed period, at any time of the day. The concentration of the hCG hormone is usually higher in the first morning urine.

3. I have an irregular period. How do I know when to do the test?

If your period is irregular, you can use Surecheck[®] 19 days after you think you may have conceived. If the result is negative, you may test again in 48 hours.

4. What should I do if the line in the test region (T) is very pale?

Test again using a new Surecheck[®] Streamer test kit after 48 hours. The color intensity of the test line may vary due to changes in the concentration of hCG hormone.

5. What circumstances may affect the result?

- Certain medications such as Humegon, Pregnyl, Profasi, Pergonal, or A.P.L. may contain hCG hormone; their intake may affect the test result.
- If Surecheck[®] is used within the first 8 weeks following childbirth or abortion, a false positive result is possible because of the residual concentration of the hCG hormone in the body. If concerned, please ask a doctor to interpret the result.
- The intake of large amounts of liquids may lower the concentration of the hCG hormone in the urine and thus affect the result.

Birth control pills, antibiotics, analgesics (painkillers) and alcohol will not affect the result.

Each Surecheck[®] Streamer box contains: foil pouch with one sterile Surecheck[®] Streamer pregnancy test; desiccant (moisture absorbing pouch; do not open!); user information leaflet; calendar. Blue Cross Bio-Medical Co. Ltd., Beijing Industry Zone, PRC 102209. Shanghai International Holding Corp. GmbH (Europe), Hamburg, Germany. Keep out of reach of children. Not for internal use. A single-use self-test. In vitro diagnostic device for home use. Test sensitivity: 25 mIU/ml. Keep away from direct sunlight, heat and moisture. **Store at a temperature between 4°C and 30°C.**

Importer and Distributor: UniComs Bulgaria Ltd., Abacus Business Centre, Floor 5, 118 Bulgaria Boulevard, Sofia 1618, Bulgaria, T +359 2 854 85 25, E office@unicoms.com

Surecheck[®] is produced for
UniComs Switzerland GmbH.



www.surecheck.info